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Visitez la page web du Varroa EasyCheck

INSTRUCTIONS FOR USE 

Varroa EasyCheck is a practical tool for easily monitoring varroa mites infestations in adult bees.

With the **CO₂ method**, bees and varroa mites are sedated by exposure to carbon dioxide.

Advantages : A quick and easy method that preserves the bee sample.

INSTRUCTIONS FOR USE

Required material (not included) : Varroa EasyCheck + 16 g CO₂ cartridges, non-threaded (pierce type)

- 1**

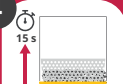
Step 1 : Place the CO₂ cartridge inside the injector and screw it in tightly to pierce the cartridge and prevent leaks. Collect a **sample of 200 or 300 bees** with the white basket of the **Varroa EasyCheck**, preferably from a frame of capped brood (**make sure the queen is not in the sample**). Place the basket back upside-down into the transparent bowl and push it until it's correctly seated inside the bowl.
- 2**

Step 2 : Place the yellow lid on the top. Create a small opening between the yellow lid and the transparent container and inject the CO₂ through a hole of the white basket for 5 to 6 seconds, until the bees stop flying. **Do not inject directly toward the bees.** Then, quickly screw the lid. Let the Varroa EasyCheck set for about 10 seconds, until the bees are anesthetized.
- 3**

Step 3 : Turn the Varroa EasyCheck **upside-down and gently shake** the sample for 15 seconds, so you do not hurt the bees. Varroa mites will be dislodged from bees and fall through the holes of the basket.
- 4**

Step 4 : Open the yellow lid and **count the mites** inside. Put the bees back in the hive where they will recover. Dispose of the fallen varroa mites, as most will still be alive.

Visit the Varroa EasyCheck website

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