3 in 1

Alcool
Alcohol

Sucre
Sugar
Azúcar

CO₂

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Sucré
Sugar
Azúcar

CO₂

Varroa EasyCheck is a simple tool for easily monitoring varroa infestation on adult bees.

It now offers **three control methods**, adapted to different beekeeping practices.

Advantages : fast and reliable diagnosis, efficient separation of varroa mites/bees, and depending on the method - preservation of the live bee sample.

The 3 methods :

- **Alcohol wash**
- **Powdered sugar roll**
- **CO₂ injection**

INSTRUCTIONS FOR USE

Alcohol wash

Required equipment (not included): denatured alcohol, diluted ethanol or windshield washer fluid containing alcohol.

Step 1: Pour the liquid up to halfway in the transparent container.

Step 2: Collect a sample of 200 to 300 bees using the white basket from a frame of capped brood (do not take the queen). Insert the basket upside-down into the bowl and press it firmly so it locks in place, it should not fall out when turning the EasyCheck over. Close with the yellow lid.

Step 3: Gently shake the Varroa EasyCheck® for 1 minute to dislodge the mites.

Step 4: Count the mites visible at the bottom of the container. The liquid can be filtered and reused up to 10 times for additional mite counts.

Powdered sugar roll method

Required equipment (not included) : powdered sugar + a container to collect the mites.

Step 1 : Place two heaping tablespoons of powdered sugar into the transparent bowl.

Step 2 : Collect 200 ou 300 bees using the white basket from a frame of capped brood (do not take the queen). Insert the basket upside-down into the bowl and press it firmly so it locks in place (it should not fall out when turning the EasyCheck over). Close with the yellow lid.

Step 3 : Gently roll the Varroa EasyCheck® for 1 minute, then let it sit for 3 minutes to allow the mites to separate from the bees.

Step 4 : Remove the lid, invert the Varroa EasyCheck® and shake it over the yellow lid or a larger tray. Add a little water to dissolve the sugar and count the mites. Release the bees back into the hive or at the entrance, they will recover normally.

Injection de CO₂

Required equipment (not included) : CO₂ applicator + CO₂ cartridge.

Step 1 : Collect 200-300 adult bees using the white sampling basket, ideally from a capped-brood frame (do not take the queen). Insert the basket upside-down in the bowl and press it down to lock it in place.

Step 2 : Place the yellow lid on top, leaving a small gap to insert the CO₂ nozzle. Inject CO₂ for 5-6 seconds through the holes of the basket — just enough for the bees to become immobilized. Do not spray directly onto the bees. Quickly secure the lid and allow the bees 10 seconds to become fully anesthetized.

Step 3 : Turn the Varroa EasyCheck upside down and gently shake for 15 seconds to dislodge the mites without injuring the bees. The mites will fall through the basket grid.

Step 4 : Open the yellow lid and count the mites collected inside. Return the anesthetized bees to the hive; they will wake up shortly. Dispose of the mites, as most will still be alive.

Visit the Varroa EasyCheck webpage

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